

A smile *you'll love* starts today!

Everything you need to *get started* with CandidPro clear aligners.

Five simple steps to a *straighter, healthier* smile



1 SHOW US YOUR TEETH

Your doctor will take diagnostic records to determine if you're a candidate.



2 BEGIN TREATMENT

You'll receive a kit with everything you need to achieve your dream smile.



3 WEAR ALIGNERS

Wear your aligners as prescribed to gently shift your teeth over time.



4 TRACK PROGRESS

Use ProMonitoring™ app to share updates with your doctor and see your new smile come to life.



5 SHOW US YOUR RESULTS

Come back for a check-in when you've completed treatment.



A quick CandidPro Q&A

WILL I NEED TO SEE MY DOCTOR FOR FOLLOW-UP APPOINTMENTS?

You'll share progress updates with your doctor digitally, so most of your treatment can be completed from home. If they spot an issue when reviewing your progress updates, they'll ask that you come in to assess the issue in person.

HOW DOES PROMONITORING™ APP WORK?

You'll download ProMonitoring™ app to your smartphone and use a ScanBox Pro (included with your treatment) to share photos of your smile with your doctor every few weeks. It's a simple process that can be completed from anywhere life takes you.

WHAT IF I NEED HELP?

If you lose an aligner or aren't sure what to do, use the live chat feature in ProMonitoring™ app for advice on what to do next.

WHAT HAPPENS AT THE END OF TREATMENT?

If you and your doctor are happy with your results, your doctor will likely prescribe a retention device or additional aligners if your smile still needs some improvement.

What's in my CandidPro *kit*?



ALIGNERS AND CASE

Clear plastic trays made just for you and a case to keep them safe.



WHITENING FOAM

Gentle, refreshing whitening foam to help keep your smile glowing.



SCANBOX PRO

Works with your smartphone to share progress updates with your doctor.

Have additional questions?

Ask your doctor or hygienist for more information about CandidPro clear aligners.

